

OSTENIL® is an isotonic solution of highly purified hyaluronan (sodium hyaluronate) developed specifically for the treatment of osteoarthritis.



Long-term relief in OA

✓ Reduce Pain


Contains 20mg / 2ml sodium hyaluronate and is administered intra-articularly as a course of three to five weekly injections.

The beneficial effects of OSTENIL® develop gradually during a course of injections and persist for several months. Following a course of 3-5 injections, symptomatic improvement can be expected to last for up to 12 months.

Improve Function

Effect of Sodium Hyaluronate Injections

What happens in the synovial joint after an injection with sodium hyaluronate?

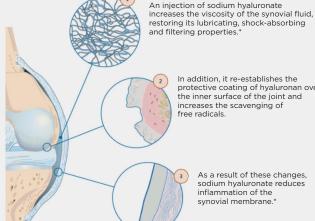

In addition, it re-establishes the protective coating of hyaluronan over the inner surface of the joint and increases the scavenging of free radicals.

> As a result of these changes, sodium hyaluronate reduces inflammation of the

OSTENIL® Plus, like the standard version of OSTENIL®, is an isotonic solution of highly purified sodium hyaluronate (or hyaluronic acid) developed specifically for the treatment of osteoarthritis.

The novel formulation of OSTENIL® Plus utilises the combination of high concentration of sodium hyaluronate + mannitol. This formulation offers the option to reduce the number of SH injections to only one injection per treatment cycle.



20mg / 2ml

OSTENIL® provides long-lasting pain relief and improves function in knee OA patients, with excellent tolerability

- Treatment cycle of 3-5 weekly injections into large joints.
- In patients suffering from hip OA, a treatment cycle of 3 weekly injections of OSTENIL® is effective in reducing pain over 12 months compared to lidocaine.
- Reduces pain and improves function in patients with advanced OA of the shoulder, who either refuse or are considered medically unfit for shoulder replacement surgery.



One injection of OSTENIL® PLUS reduces pain and improves function in knee OA over a period of at least 6 months.

- Mannitol-stabilised high dose HA 2%.

 Mannitol acts as a free radical scavenger

 which protects HA from rapid depolymerisation.
- The formulation offers the possibility of increasing the intervals between injections
- Sustained efficacy with reduced number of injections (1-3).

Distributed by:



- **Q** 02 673 4781 / 02 658 8717
- info@propharma.ae
- www.propharma.ae